

Lucas Service Center IR Thermometer

Veridian 09-349

0. Gloves must be worn for forehead temperature check.

1. Press the **Power** button to turn the thermometer on. The thermometer will beep when ready.

2. Press and hold the **MEASURE** button.

3. While holding the **MEASURE** button, aim the infrared lens at the patient's forehead, holding the thermometer approximately 2.5 – 5 cm away. Ask subject to hold hair out of the way if necessary. The thermometer may begin to beep.

4. Release the **MEASURE** button, **DO NOT MOVE THE THERMOMETER; THE MEASUREMENT IS NOT YET COMPLETE.**

5. The thermometer will beep a final time when through reading, and the temperature will be displayed. Record the temp reading on the screening form.

6. The last 10 readings taken are recalled in order by depressing the **MEM** button.

7. The thermometer should be wiped with alcohol prep pad after use; do not use water.

About one minute after use, the thermometer will automatically beep and shut off, else the **Power** button can be pressed.

